

Artmex V7

MTS & PMU System




User Manual



Contents

Product detail	1
Basic operation	3
Intelligent hand-piece	9
Needles head installation	10
Adjustment of needles depth	10
Needles head introduction	11
Specification	12
Precaution	13

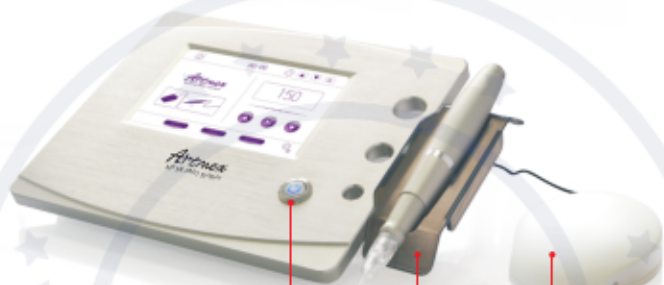

① Power switch

② Holder


③ Pedal

④ Power interface

⑤ Hand piece interface

⑥ Pedal interface



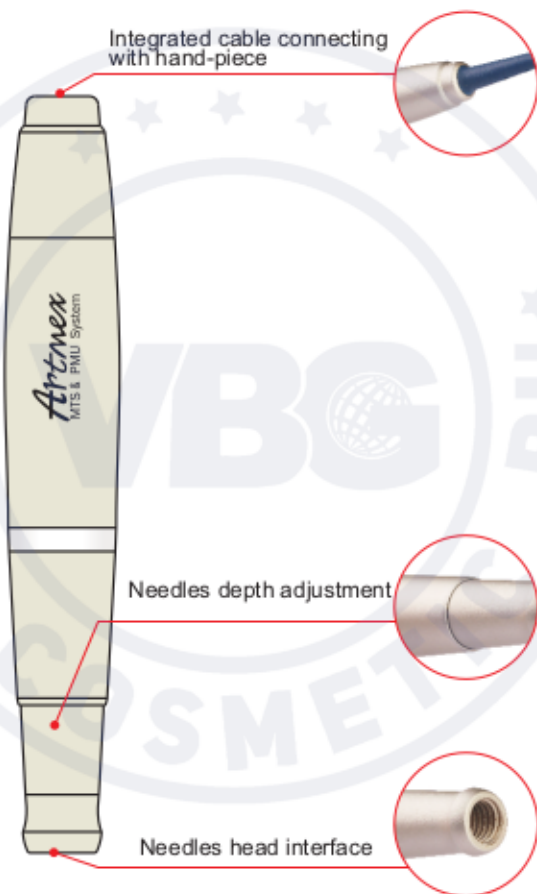

PMU marks the color pigments inhale into the needle by injecting them to skin by a back-and-forth motion

MTS helps medical supplies be absorbed to skin by irritating

Basic Operation

1. Place accessories well and connect to main body



Power

Fig 1

Fig 2

Cord of adapter connect to power interface
(Operating like fig 1 and fig 2)



Pen

Fig 3

Fig 4

Cord of hand piece connect to hand piece interface
(Operating like fig 3 and fig 4)



Pedal

Fig 5

Fig 6

Cord of pedal connect to pedal interface
(Operating like fig 5 and fig 6)

2. Plug DC charger to AC outlet
3. Press the power switch for 2 seconds to enter the loading interface.
It will enter the interface as (Fig 7) after finish loading interface.



Fig 7

4. Click the interface(Fig 7) to enter interface as(Fig 8)

The hand-piece icon will flash when the host without connecting hand-piece

The function key is unselectable when the hand-piece icon is flashing.

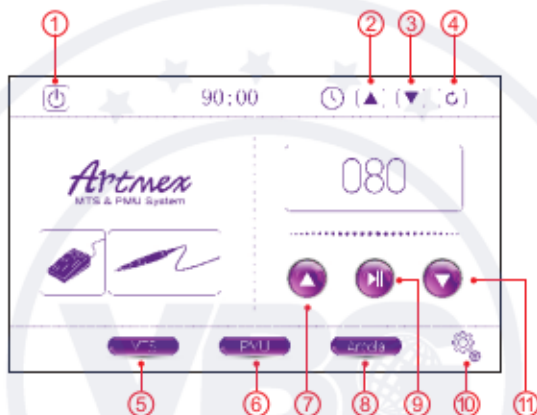


Fig 8

- | | |
|----------------------|-----------------------|
| ① Shutdown Button | ② Working Hours Up |
| ③ Working Hours Down | ④ Reset Working Hours |
| ⑤ MTS Mode | ⑥ PMU Mode |
| ⑦ Speed Up | ⑧ Areola Mode |
| ⑨ Start/Pause | ⑩ Set Default Speed |
| ⑪ Speed Down | |

5. Mode selection(MTS&PMU&AREOLA)

[1] Click "  " to enter the interface as(Fig 9)



Fig 9

[1] Click the dashed region of face to choose the treated part.

[2] Click  or  to select the speed level

21 speed levels. Higher level, higher speed

[3] Click  to start or pause the hand-piece

Or use pedal to control the hand-piece.

[4] Adjust the needles depth(Maximal depth is 3.0mm)

Needles depth adjustment should be done while the pen working

6. Click "PMU" to enter the interface as(Fig 10)



Fig 10

The operation of PMU is the same as MTS after choosing the treated part.
-Click the solid line region of face to choose the treated part.
(include the eyebrows & eyeliner & lip)

7. Click "Areola" to enter the interface as(Fig 11)

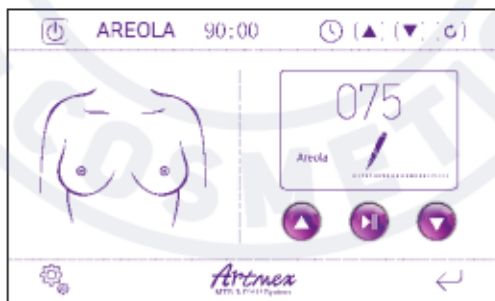


Fig 11

The operation of AREOLA is the same as MTS and PMU


8. Click “” to enter the interface as(Fig 12)



Fig 12

set default speed for each function.

[1] Select function.

[2] Click “up” or “down” to adjust the speed.

[3] Click “ok” finish adjusting the default speed.

Use the stylus to touch the screen



Stationary mode



The pen will stop working automatically when it stays on the placement area.

Operating mode



The pen will start working automatically when it leaves the placement area.

8 (800) 222-03-94
8 (495) 532-03-23

Приём звонков:
Пн-Пт с 9:00 до 18:00



<https://vbg-cosmetic.ru/>