

## About us

Evie is a company that manufactures and sells Cosmeceutical products to women who seek beauty. We first launched in Korea in 2006 and now currently export our products to various countries such as China, Indonesia, Malaysia, Vietnam, Singapore, and Russia.

Evie's representative product is the MTS Ampoule. Our MTS (Microneedle Therapy System) is a cosmetic procedure that treats underlying skin damage using a hair fine needle. Incorporating the principle of skin rejuvenation, MTS is a treatment that brings the skin back to its original state by stimulating damaged skin using an MTS machine that has hundreds of needles.

MTS is effective for treating scars, increasing skin elasticity, reducing pores, brightening skin, and improving the appearance of wrinkles. These type of Cosmeceutical products have not only gained popularity in Korea, but also all over the rest of the world. As a result, Evie has recently started to export the MTS ampoule internationally.

One of Evie's most popular MTS products for sale is our BB Glow Ampoule. This product has been manufactured in three different colors accordingly to skin tone, but recently two new colors have also been added. This product's combination of peptides and foundation is effective for brightening dark or dull looking skin. When applied 3-4 times a week you will notice a more even skin tone. Evie also manufactures and sells a wide range of products to satisfy the needs of women who seek beauty, such as lifting packs in addition to the MTS ampoule.

We encourage you to choose Evie if you are looking for Cosmeceutical products that are enjoyed by people all over the world. Evie's product competitiveness has lead numerous international customers to repurchase our products and we hope that this will instill confidence in our future customers as well.



EVIE

# MTS

(Micro Needle Therapy System)

**The hottest aesthetic trend in Korea**

MTS is a treatment that helps to repair the skin by using small needles to stimulate fibroblast caused by physical damage to induce collagen production while allowing medicine with high molecular weight or skin products to effectively penetrate the skin.

EVIE



# What is MTS?

(Micro Needle Therapy System)


You can use the MTS roller depending on the diameter or length of the needle. In Korea, long needles about 1.0~2.0mm are used in dermatology for quick effects and require anaesthetic cream before treatment. For cases such as home skincare and aesthetic purposes, a short needle about 0.20~0.25mm is used and does not require anesthetic cream.

Why you should use the MTS roller when applying an ampoule: Skincare products stay stuck in the stratum corneum and are unable to penetrate completely into the skin.

About only 0.03~2% of skincare products are absorbed into the skin, and the rest remains on the outer layer of the skin and is washed away.


In order to increase the absorption rate of skincare products, we use a needle to administer the active ingredient directly into the skin.

[Cosmetics alone]

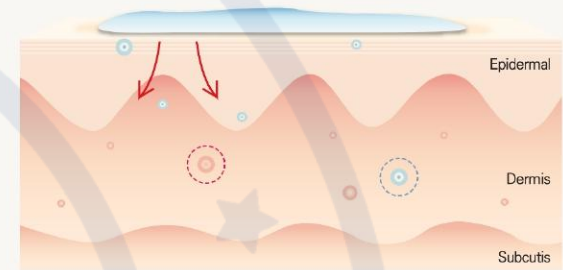

Even in highly concentrated skincare products, the amount of effective ingredients absorbed into the outer layer of the skin is miniscule.

[Cosmetics & MTS]

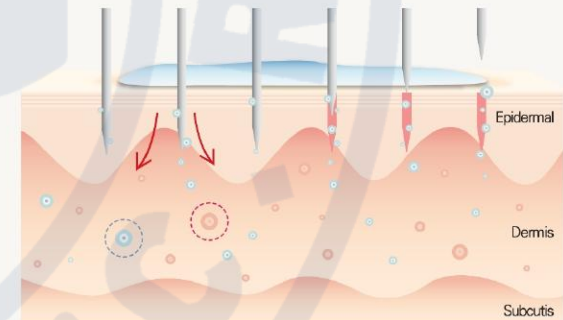

The effective ingredient of skincare products absorbed to the outer layer of skin is enhanced by delicate needling and then slowly absorbed into the dermis layer, thereby promoting collagen production.

## Dermawhite Gold Peptide Ampoule

A mixture of various peptide ingredients and 99.9% pure gold powder. Cleanses, removes toxins, and clears the skin. Promotes the skin regeneration cycle. Whitening effect.

8ml\*10pcs



stayve

## Exfoliating gel

Thanks to synergic combination of shikimic and mandelic acid, gel relaxes inter-cellar. Stimulates regeneration of deeper skin layers. Simultaneously, it activates fibroblasts in dermis, increases synthesis of collagen and elastin, improving skin appearance. Brightens existing discoloring and makes skin color even. The skin becomes soft and silky smooth.

220ml



## Neutralizing foam

Stopping the operation of acids, restoring proper PH to the skin, alleviation of irritations, reduction of redness, preparation of skin for next stages of treatment.

150ml



## Three types of stem cell culture ampoule (whitening & wrinkles, moisture, acne)

Contains stem cell culture fluid ingredients and various active ingredients to make skin smooth and clear. Clear water-soluble ingredient can be used for home care after treatment as well as for MTS treatment for maximum effectiveness.



## Repair cream

A special cream that contains lanolin and bio-peptide that improves overall skin texture rather than maintaining original skin condition. Promotes skin regeneration when frequently applied to customers right after laser treatments, semi permanent treatment, or the MTS procedure.

100pcs\*1g / box

stayve



## Whitening & lifting pack

Whitening & lifting pack(6ml\*10pcs)

Whitening & lifting polymer(1ml\*10pcs)

Peptide cream(30g)

## Machine used for Semi permanent and MTS procedures

Absorbs and removes fat and toxins from the skin and has a powerful lifting and wrinkle-reducing effect. Cleansed skin allows you to transfer various high-functional skincare ingredients directly into your skin. Process gives excellent whitening effect and skin glow that leaves you with bright, clear and beautiful skin.

9 adjustable stage



# HOW TO USE

## KOREAN BB GLOW TREATMENT



1. Thoroughly cleanse your face.
2. Lightly apply toner to the face with a cotton pad.
3. Using a brush, apply a thick layer of the exfoliating gel. The MTS Treatment is more effective when used after exfoliating. It is not recommended to use any other exfoliating gel besides the one included in the MTS Treatment. The acidity in the Stayve exfoliating gel compliment the MTS BB Glow Treatment. Do not rub the product in with your hands after applying the exfoliating gel.

Dry skin: 2 minutes Normal skin: 3 minutes Oily skin: 5 minutes  
The acidity in the Stayve exfoliating gel may slightly sting on sensitive skin.

4. Wash your face or rinse with a damp towel or sponge.
5. Apply the neutralizing foam with a brush. This foam takes away the acidity that occurs with the exfoliating gel. Softly massage with your hands for 3 minutes. Wash your face or rinse with a damp towel or sponge.
6. Begin the MTS Treatment (for 2-3 minutes) after applying the booster ampoule. The booster ampoule comes in various types depending on your skin's needs. Including, acne, lifting, whitening, and aqua. You can apply the regular BB ampoule instead of the booster, but the booster ampoule procedure is more effective when used with the BB Glow Treatment.
7. Begin the MTS Treatment (for 2-3 minutes) after applying the BB GLOW ampoule.
8. Use a mask pack or modeling pack if you experience any irritation. You can also use the Omega Light machine for 10-15 minutes. This step can be skipped if you don't have the machine.
9. After removing the mask, cleanse your face with a damp sponge and apply the Stayve Repair Cream.
10. We provide 10 additional Repair Creams that you can use everyday after cleansing. We recommend that you sell the vitamin ampoule for whitening etc. to customers for maximum effect during the After treatment.

# Needle length of MTS Machine



## 1. Needle length of MTS Machine

The following type of needles are included in the MTS procedure. Nano needle – Very small needle, painless but least effective Middle needle (0.2~0.25mm) – Middle sized needle, most recommended length, does not require Anesthesia cream Areas such as the nose or forehead may be more painful but we do not recommend using the anesthesia cream. More painful but effective than the nano needle. Long needle (1.5~2.0mm) – Needles used in hospitals, very long and the most painful so it requires anesthesia cream. However, the most effective. Depending on your skin type, you may experience various side effects.

## 2. How to apply the Anesthesia cream

Apply to the whole face using a brush, making sure to avoid the nose and mouth, and then wrap. Wait 20-30 minutes before wiping it off and then continue with the MTS Treatment. When using the 0.25mm needle, it is not recommended to apply the Anesthesia cream, even if painful. The anesthesia cream can cause the pores to shrink, causing the BB Ampoule not to absorb properly. Areas with more muscle such as the cheeks are not as painful as areas with bone (nose and forehead). We recommend that you continue with the procedure more softly on sensitive areas.



## 3. After Treatment

The skin becomes dry and flaky 3-4 days after using the long needle. Moisturization and repair steps are especially important when using the long needle. Apply the Vitamin Ampoule, Repair Cream, and Booster Ampoule every morning and night as well as sunscreen after the procedure.

## 4. Procedure period

The treatment is most effective when done regularly, once a week, for about 4-5 weeks. The results with the nano needle last about 2-3 months, and results with the 0.25mm needle last more than 3-4 months.



## HOW TO USE STAYVE WHITENING & LIFTING PACK

**Case 1.** Wash your face and dry with a towel, leaving your face bare.

**Case 2.** Open the Lifting Pack up until the dotted line, and add in the polymer while gently shaking for 30 seconds.

**Case 3.** Shaking the product for 30 seconds will cause the thin liquid to turn into a gel consistency.

**Case 4.** Apply to your face with a brush, starting from the bottom and moving upwards.

**Case 5.** Leave the product on for 20-30 minutes. To speed up the process, you can use a fan.

**Case 6.** Rinse the product off with a damp towel or by washing your face with warm water.

**Case 7.** Finally, apply the peptide cream to complete the treatment.



### Before & After ...





ООО «Восток Бизнес Групп»

Адрес: г. Москва, Бизнес-парк «Румянцево» 22-й км  
Киевского шоссе, ДВЛД4, стр.2, корпус Г, оф.509

Тел. 8 (800) 222-03-94

Тел. +7 (495) 532-03-23

(Прием звонков с 9-00 до 18-00)

Сайт [vbg-cosmetic.ru](http://vbg-cosmetic.ru)

