

Artmex

MTS & PMU System


VG




User Manual



No.	Description
1	Holder
2	Power interface
3	Hand piece interface
4	Adapter
5	Cable

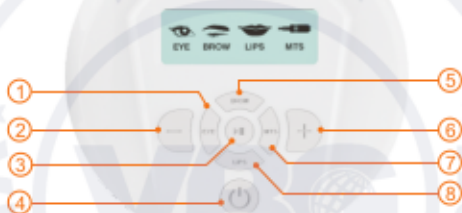

No.	Description
1	Eye button
2	Speed adjustment (down)
3	Start/pause button
4	Power switch
5	Brow button
6	Speed adjustment (up)
7	MTS button
8	Lip button

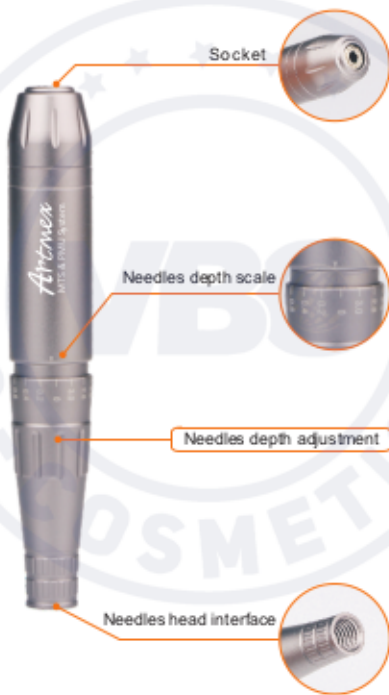

PMU marks the color pigments inhale into the needle by injecting them to skin by a back-and-forth motion
MTS helps medical supplies be absorbed to skin by irritating

Basic Operation

1.Place accessories well and connect to main body



Fig 2



Fig 3

Cord of hand piece connect to hand piece interface
(Operating like fig 2 and fig 3)



Fig 4



Fig 5

Cord of adapter connect to power interface
(Operating like fig 4 and fig 5)

2. Plug DC charger to AC outlet

3. Click  to turn on the machine and it show like Fig 6



Fig 6

4. Choose the mode.

Click  to enter the interface as Fig 7

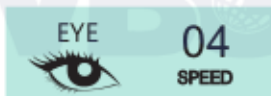


Fig 7

The speed starts from Level 4

Click  to enter the interface as Fig 8

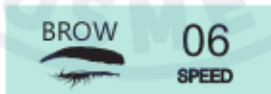


Fig 8

The speed starts from Level 6

Click  to enter the interface as Fig 9



Fig 9

The speed starts from Level 6

Click  to enter the interface as Fig 10



Fig 10

The speed starts from Level 8

5. Install the needles cartridge. Selectable disposable needle cartridge between MTS and PMU for its purpose

6. Click   to select the speed level

10 speed levels. Higher level, higher speed

7. Click  to make the pen start working

8. Adjust the needles depth (Maximal depth is 3.0mm)
Needles depth adjustment should be done after the pen working


- (1) Take out needles cartridges in the sterilized packing
- (2) Pushing the needle cartridge horizontally into the round slot of pen
- (3) Assemble the needles cartridges by clockwise rotation
Take apart the needles cartridges by counterclockwise rotation
- (4) Adjustment of needle length should be done after turn on the machine



Adjustment of needles depth

Adjusting the depth of needle by turning the needles depth adjustment. Turn it clockwise, needle depth will be longer. Turn it counterclockwise, needles depth will be shorter



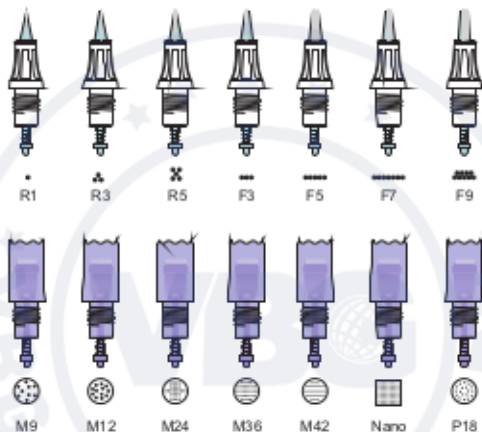

Package List

Item	Quantity
Host	1 pc
Hand piece	1 pc
Holder	1 pc
Adapter	1 pc
Cable	2 pcs
Cartidges	5 pcs






















Product information	
Classification	Contents
Model name	ARTMEX-V6
Patented product	Yes
Purpose of Use	Permanent make-up & Micro-needle Therapy
Aadpter	100V~240V
Weight	860g
Size	247mm*190mm*76mm


Maintenance
1.General maintenance

- Keep it with its power off. Wipe its surface out with a dry cloth.
- Keep it away from direct sunlight or high temperature and humidity.
- Do not leave the measurement, repair or alter of the device to non-professional staff.

2.Routine inspection

- Make sure to inspect the device and adapter before use.
- When you use the device which has not been used for a long time, make sure to check if it works properly with safety.

	Alternating current		Serial No
	Direct current		EU Representative
	Class II equipment		CE mark
	Attention, consult accompanying documents		Manufacturer
	Date of manufacture		Floating applied part
	Please be sure to read instructions before use and follow as described.		
	Do not apply it to the man as follow: 1. Who has acute disease or bloody disease over 38℃ 2. Who has allergy and hemophilia 3. Who has melanosis of the face 4. Who has high internal heat 5. Who regularly visits hospital 6. Who takes medicines 7. Who is under 12 years old		
	Do not use it on the power other than AC 220V. It may cause fires or malfunction		
	Do not use it near the place with high temperature, dust or humidity (Near the humidifier). It may cause malfunction.		
	Do not repair, disassemble or alter the controller and the body. It may cause malfunction.		
	Do not touch the device or detach the power cords with wet hands. It may cause electric shock.		
	Do not use the stripped cord or propped cord. It may cause fires(Be sure to replace it with another).		
	Do not use the propped cord which has crack or scars. It may cause scars on the skin.		
	When you don't use it for a long time, unplug it from the outlet.		
	Be sure to have an inspection for test. If any troubles are found, get it repaired.		
	No one other than the operator can touch it. It may cause accidents or malfunction.		

8 (800) 222-03-94
8 (495) 532-03-23

Приём звонков:
Пн-Пт с 9:00 до 18:00

Artmex
User Manual

<https://vbg-cosmetic.ru/>