

Artmex V7

MTS & PMU System



User Manual



Contents

Product detail	1
Basic operation	3
Intelligent hand-piece	9
Needles head installation	10
Adjustment of needles depth	10
Needles head introduction	11
Specification	12
Precaution	13



① Power switch

② Holder


③ Pedal

④ Power interface

⑤ Hand piece interface

⑥ Pedal interface



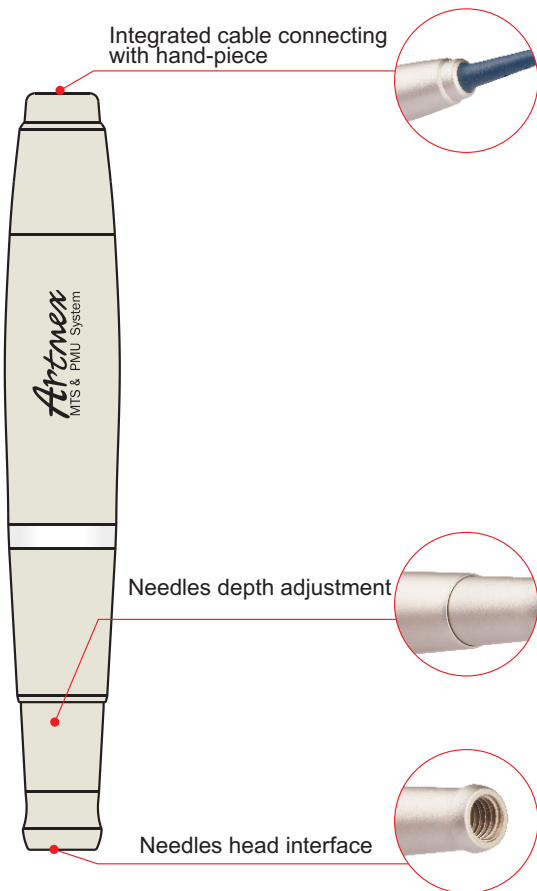

PMU marks the color pigments inhale into the needle by injecting them to skin by a back-and-forth motion


MTS helps medical supplies be absorbed to skin by irritating

Basic Operation

1. Place accessories well and connect to main body



Fig 1

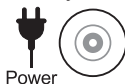

Power


Fig 2

Cord of adapter connect to power interface

(Operating like fig 1 and fig 2)



Fig 3

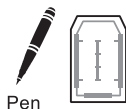

Pen


Fig 4

Cord of hand piece connect to hand piece interface

(Operating like fig 3 and fig 4)



Fig 5

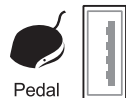

Pedal

Fig 6

Cord of pedal connect to pedal interface

(Operating like fig 5 and fig 6)

2.Plug DC charger to AC outlet

3.Press the power switch for 2 seconds to enter the loading interface.

It will enter the interface as (Fig 7) after finish loading interface.



Fig 7

4.Click the interface(Fig 7) to enter interface as(Fig 8)

The hand-piece icon will flash when the host without connecting hand-piece

The function key is unselectable when the hand-piece icon is flashing.

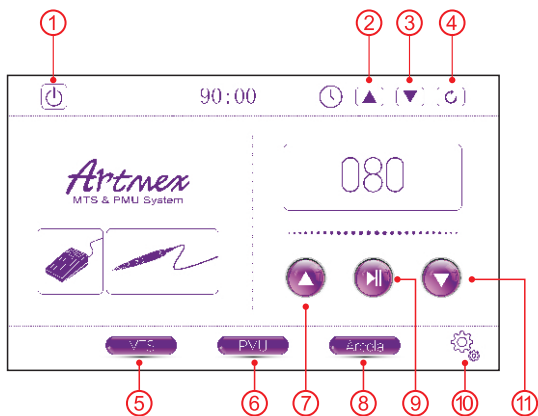


Fig 8

- | | |
|----------------------|-----------------------|
| ① Shutdown Button | ② Working Hours Up |
| ③ Working Hours Down | ④ Reset Working Hours |
| ⑤ MTS Mode | ⑥ PMU Mode |
| ⑦ Speed Up | ⑧ Areola Mode |
| ⑨ Start/Pause | ⑩ Set Default Speed |
| ⑪ Speed Down | |

5. Mode selection(MTS&PMU&AREOLA)

[1] Click “ MTS ” to enter the interface as(Fig 9)

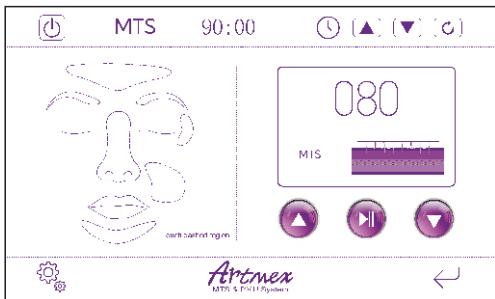





Fig 9

[1] Click the dashed region of face to choose the treated part.

[2] Click  or  to select the speed level
21 speed levels. Higher level, higher speed

[3] Click  to start or pause the hand-piece
Or use pedal to control the hand-piece.

[4] Adjust the needles depth(Maximal depth is 3.0mm)

Needles depth adjustment should be done while the pen working

6. Click “ **PMU** ” to enter the interface as(Fig 10)

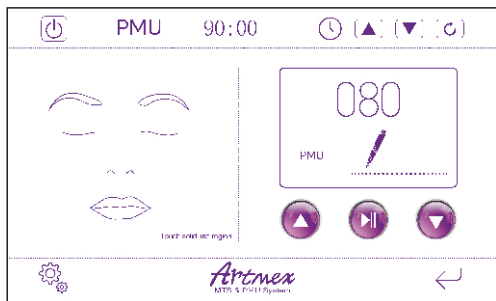


Fig 10

The operation of PMU is the same as MTS after choosing the treated part.
 -Click the solid line region of face to choose the treated part.
 (include the eyebrows & eyeliner & lip)

7. Click “ **Areola** ” to enter the interface as(Fig 11)

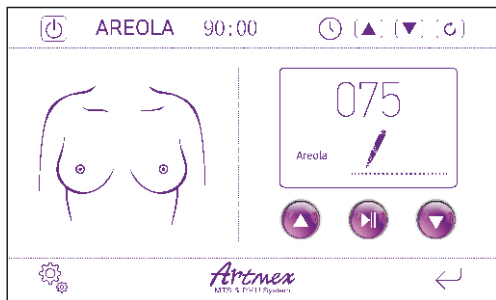



Fig 11

The operation of AREOLA is the same as MTS and PMU

8. Click “” to enter the interface as(Fig 12)

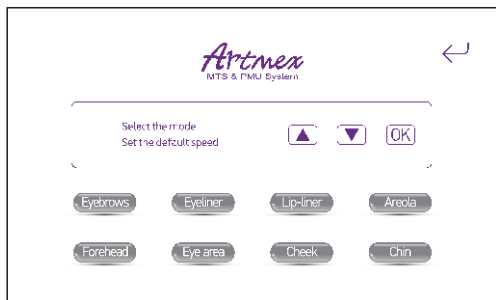


Fig 12

set default speed for each function.

[1] Select function.

[2] Click “up” or “down” to adjust the speed.

[3] Click “ok” finish adjusting the default speed.

Use the stylus to touch the screen




Stationary mode




The pen will stop working automatically when it stays on the placement area.

Operating mode

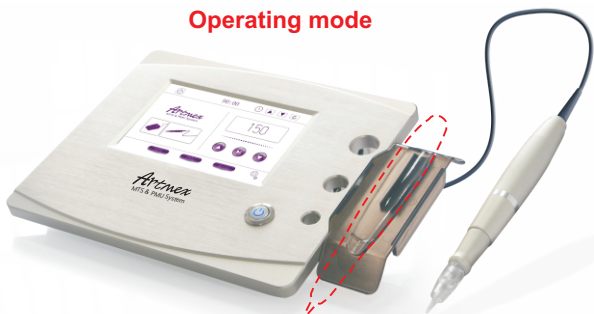

The pen will start working automatically when it leaves the placement area.

1. Take out needles cartridges from the sterilized package.
2. Pushing the needle cartridge horizontally into the round slot of pen.
3. Assemble the needles cartridges by clockwise rotation take apart the needles cartridges by counterclockwise rotation.
4. Adjustment of needle length should be done after turning on the machine.

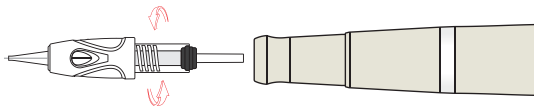


Adjustment of needles depth

Adjusting the depth of needle by turning the needles depth adjustment.

-Turn it clockwise, needle depth will be longer.

-Turn it counterclockwise, needle depth will be shorter

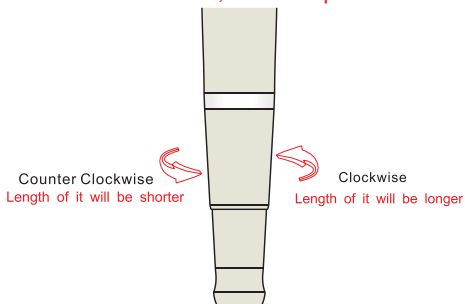



M1



L1



R3-M



R3-L



R5



F3



F4



F5



F7



F9



S3



S5



V9



V12



M9




M12



M24



M36

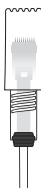

M42



Nano



P18

【Model name】 : ARTMEX - V7

【Weight】 : 2650g

【Adapter】 : 100V~240V

【Size】 : 290mm*273mm*105mm

【Patented product】 : Yes

【Purpose of Use】 : Permanent make-up &
Micro-needle Therapy

Maintenance

1.General maintenance


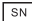















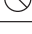



- Keep it with power off. Wipe its surface up with a dry cloth.
- Keep it away from direct sunlight or high temperature and humidity.
- Do not leave the measurement, repair or alter of the device to non-professional staff.

2.Routine inspection

- Make sure to inspect the device and adapter before use.
- When you use the device which has not been used for a long time, make sure to check if it works properly with safety.

Package list

Host	1 pc
Holder	1 pc
Adapter	1 pc
Hand piece	1 pc
Stylus	1 pc
Pedal	1 pc

	Alternating current		Serial No
	Direct current		EU Representative
	Class II equipment		CE mark
	Attention, consult accompanying documents		Manufacturer
	Date of manufacture		Floating applied part
	Please be sure to read instructions before use and follow as described.		
	<p>Do not apply it to the man as follow:</p> <ol style="list-style-type: none"> 1. Who has acute disease or bloody disease over 38°C 2. Who has allergy and hemophilia 3. Who has melanosis of the face 4. Who has high internal heat 5. Who regularly visits hospital 6. Who takes medicines 7. Who is under 12 years old 		
	Do not use it on the power other than AC 220V. It may cause fires or malfunction		
	Do not use it near the place with high temperature, dust or humidity (Near the humidifier). It may cause malfunction.		
	Do not repair, disassemble or alter the controller and the body. It may cause malfunction.		
	Do not touch the device or detach the power cords with wet hands. It may cause electric shock.		
	Do not use the stripped cord or propped cord. It may cause fires (Be sure to replace it with another).		
	Do not use the propped cord which has crack or scars. It may cause scars on the skin.		
	When you don't use it for a long time, unplug it from the outlet.		
	Be sure to have an inspection for test. If any troubles are found, get it repaired.		
	No one other than the operator can touch it. It may cause accidents or malfunction.		



Artmex
MTS & PMU System